

TO 00-35A-39

---

**TECHNICAL MANUAL**

**INSTRUCTIONS FOR PROCUREMENT, ISSUE, USE AND  
MAINTENANCE OF MEDICAL KITS**

(ATOS)

THIS PUBLICATION SUPERSEDES TO 00-35A-39, DATED 1 APRIL 2001.

DISTRIBUTION STATEMENT: Approved for public release; distribution is unlimited.

Published under authority of the Secretary of the Air Force

---

**15 NOVEMBER 2002**

**CHANGE 2 - 15 SEPTEMBER 2004**

# LIST OF EFFECTIVE PAGES

NOTE: The portion of the text affected by the changes is indicated by a vertical line in the outer margins of the page. Changes to illustrations are indicated by miniature pointing hands. Changes to wiring diagrams are indicated by shaded areas.

**Dates of issue for original and changed pages are:**

Original ..... 0 ..... 15 November 2002    Change ..... 2 ..... 15 September 2004  
 Change ..... 1 ..... 1 April 2004

**TOTAL NUMBER OF PAGES IN THIS MANUAL IS 40, CONSISTING OF THE FOLLOWING:**

Page No.	*Change No.	Page No.	*Change No.	Page No.	*Change No.
<b>Title</b> .....		<b>2</b>			
<b>A</b> .....		<b>2</b>			
i - ii .....		1			
1-1 - 1-7 .....		1			
<b>1-8</b> .....		<b>2</b>			
1-9 - 1-16 .....		1			
<b>1-17</b> .....		<b>2</b>			
1-18 Blank .....		1			
1-19 - 1-21 .....		1			
<b>1-22</b> .....		<b>2</b>			
1-23 - 1-25 .....		1			
<b>1-26</b> .....		<b>2</b>			
1-27 .....		1			
<b>1-28</b> .....		<b>2</b>			
1-29 - 1-32 .....		1			
<b>1-33 - 1-35</b> .....		<b>2</b>			
1-36 Blank .....		1			

\*Zero in this column indicates an original page

## TABLE OF CONTENTS

Chapter		Page
1.1	Purpose.....	1-1
1.2	Authorization for Use and Issue Procedures.....	1-1
1.3	Controlled Substances.....	1-1
1.4	Marking of First Aid Kits.....	1-2
1.5	Kit Components.....	1-2
1.6	Serviceability Tags.....	1-3
1.7	Inspection of Medical Kits.....	1-4
1.8	Inspection By Other Than Medical Personnel.....	1-4
1.9	Funding for Replacement Components.....	1-4
1.10	Turn In of Excess Kits.....	1-5
1.11	Furnishing Kits to Air Force Contractors.....	1-5
1.12	Kit Standardization or Modification Procedures.....	1-5
1.13	NSN 6545-01-521-8511, First Aid Kit, General Purpose, Rigid Case (Figure 1-2).....	1-5
1.14	NSN 6545-01-521-8530, Survival Kit, Individual (Figure 1-3).....	1-9
1.15	NSN 6545-01-521-8528, Survival Kit, Individual, Tropical, Tactical Aircrew (Figure 1-4).....	1-12
1.16	NSN 6545-01-521-8519, First Aid Kit General Purpose, Flexible Case, Panel Mounted (Figure 1-5).....	1-16
1.17	NSN 6545-01-521-8515, First Aid Kit General Purpose, Rigid Case (Figure 1-6).....	1-21
1.18	NSN 6545-01-521-8514, First Aid Set, Shelter.....	1-25
1.19	NSN 6545-01-521-8502, First Aid Kit, Individual (Figure 1-7).....	1-27
1.20	NSN 6545-01-521-8522, Survival Kit, Individual, Tropical, Tactical Aircrew, in a Plastic Bag.....	1-31
1.21	Component Cross Reference to First Aid Kits.....	1-33

## LIST OF ILLUSTRATIONS

Number	Title	Page
1-1	AFTO Form 104 and DD Form 1574.....	1-3
1-2	First Aid Kit, General Purpose, Rigid Case, NSN 6545-01-521-8511.....	1-6
1-3	Kit Assembly.....	1-13
1-4	Survival Kit, Individual, Tropical, Tactical Aircrew, NSN 6545-01-521-8528.....	1-15
1-5	First Aid Kit, General Purpose, Flexible Case, Panel Mounted, NSN 6545-01-521-8519.....	1-19
1-6	First Aid Kit, General Purpose, Rigid Case, NSN 6545-01-521-8515.....	1-23
1-7	First Aid Kit, Individual, NSN 6545-01-521-8502.....	1-29

## LIST OF TABLES

Number	Title	Page
1-1	Special Kits.....	1-2
1-2	NSN 6545-01-521-8511 - Components.....	1-8
1-3	NSN 6545-01-521-8530 - Components.....	1-11
1-4	NSN 6545-01-521-8528 - Components.....	1-14
1-5	NSN 6545-01-521-8519 - Components.....	1-17
1-6	NSN 6545-01-521-8515 - Components.....	1-22
1-7	NSN 6545-01-521-8514 - Components.....	1-26
1-8	NSN 6545-01-521-8502 - Components.....	1-28
1-9	NSN 6545-01-521-8522 - Components.....	1-32

**TO 00-35A-39**

1-10 Component Cross Reference to First Aid Kits ..... 1-33

# CHAPTER 1

## **1.1 PURPOSE.**

- 1.1.1 To list special purpose medical first aid and/or survival kits authorized for use by USAF activities.
- 1.1.2 To describe the procedures to standardize or modify medical first aid or survival kits.
- 1.1.3 To delineate responsibilities for authorizing, requisitioning, issuing, installing, inspecting, controlling and disposing of medical kits.

## **1.2 AUTHORIZATION FOR USE AND ISSUE PROCEDURES.**

- 1.2.1 Authorization for issue of special kits. (See [Table 1-1](#) .)
- 1.2.2 In addition to the authorizations listed below, various USAF technical orders (TOs) list one or more of these kits for installation in aircraft, survival kits, or other major items. Organizations requiring issuance of medical kits not specifically authorized by a current Table of Allowance (TA) will cite the appropriate TO as the authority for issue.
- 1.2.3 Special Allowance Source Code (ASC) 041 authorizations may be granted by the Military Treatment Facility Commander (MTF) as specified in AFI 41-209, Chapter 3, Paragraph 3-12.11.
- 1.2.4 Items listed in this TO will be initially issued to using activities on a reimbursable basis by the local Medical Materiel Manager (MMM). Requests for initial issues will be submitted to the MMM through the local Equipment Management Office (EMO) for verification of TO or TA authorization. Requests for replacement kits may be forwarded directly to the MMM. Initial issue of kits to USAF contractors is discussed in paragraph [1.11](#) .
- 1.2.5 The kits listed are designed for the specific purpose and use intended and will not be modified, disassembled, issued, or used in any manner not prescribed herein, which includes for practice or play exercises. However, if required to keep flight crews mission capable, the Medical Group Commander may approve temporary substitutes after consulting with the senior flight surgeon in the Wing, or on the base, preferably an Aerospace Medicine Specialist. Kits containing these will be marked with an expiration date of 30 days, during which time permanent substitutes will be coordinated with AFMSA/SGSLX and the Air Force consultant for first aid kits. Medical kits will not be used when other medical services and supplies are readily available. The kits need to be ready and completely assembled for their intended purpose.

## **1.3 CONTROLLED SUBSTANCES.**

All narcotics and non-narcotic controlled substances have been deleted from these medical kits. Controlled substances are normally not included in medical kits. Amphetamines will not be added to medical kits under any circumstances.




1.5.2.4 Other codes may be identified by reviewing the UDR under “Special Notes” prior to the “Table of Contents.”

**1.6 SERVICEABILITY TAGS.**

1.6.1 Medical Kits listed herein (except for NSN 6545-01-521-8519, First Aid Kit General Purpose, Flexible Case, Panel Mounted) will be tagged with AFTO Form 104 (First Aid/Survival Kit Inspection Certificate). First Aid Kit, General Purpose, Flexible Case, Panel Mounted NSN 6545-01-521-8519 will be tagged with DD Form 1574 (Serviceable Tag-Material).

1.6.2 The tag or label will be marked as shown in Figure 1-1. Major commands are authorized to grant life support activities the option to delete or obliterate the inspecting activity when life support equipment requires sanitizing. The MMM or the property administrator at an aircraft contractor plant will complete this tag on initial issue of the kit to indicate that the components are up to date and will not expire prior to the next inspection date.

1.6.3 In addition to inspection information the AFTO FORM 104 is used to protect the integrity of the medical kits. Kits which have a torn AFTO FORM 104 will not be issued by life support or using activities. All such kits will be returned to the MMM for reinspection.

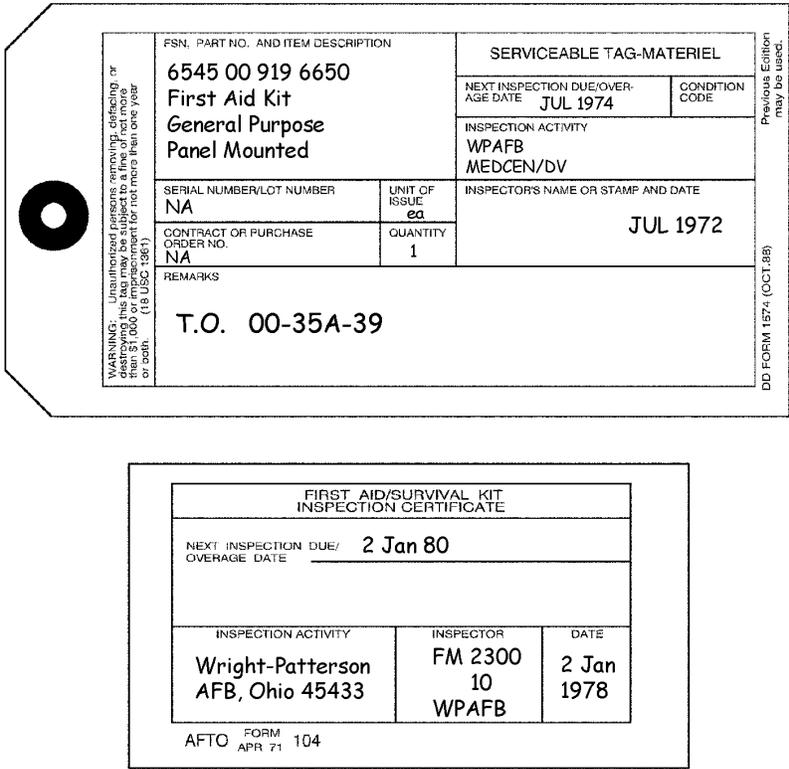

H9003172

**Figure 1-1. AFTO Form 104 and DD Form 1574**

## **1.7 INSPECTION OF MEDICAL KITS.**

1.7.1 Medical kits listed in this TO will normally be inspected by the MMM every 36 months, when suspension of the kit or one of its components is announced in the AFMLL, or when turned in by using activities in accordance with paragraph 1.8. The 36 month inspection benchmark may be extended or shortened to coincide with aircraft isochronal inspections if sufficient shelf life is available on individual components. If this procedure is followed, inspection of the kit can be included in the aircraft inspection checklist and accomplished during routine aircraft downtime. Medical logistics personnel will make every possible attempt to insure sufficient shelf life is available on kit components under this program. Using activities will insure that kits are removed from use and turned in to the MMM for inspection and quality assurance action prior to the next inspection due and/or overage date or immediately upon receiving notice from the MMM of an applicable item suspension.

1.7.2 During the inspection, the MMM will accomplish the following minimum actions.

1.7.2.1 Insure kits are properly marked. If not, marking in accordance with paragraph 1.4 will be accomplished.

1.7.2.2 Replace all expiration dated items with less than 36 months shelf life remaining wherever possible. If not possible, the NEXT INSPECTION DUE AND/OR OVERAGE DATE annotated on the serviceability tag will be that of the component that has the shortest expiration date or shelf life remaining.

1.7.2.3 Insure that the kit contains no Schedule 12s controlled medical items unless a current major command authorization is on file.

1.7.2.4 Insure the contents of the kit are complete and serviceable.

1.7.2.5 Subsistence items will be replaced by items with a later date of manufacture during normal inspection of the kits.

1.7.2.6 Insure that a list of contents and dosage instructions for FSC 6505 items are included in the kits. Local reproduction of the lists by the MMM is authorized.

1.7.2.7 Insure that DLAR-4155-37, Materiel Quality Control Storage Standards (Appendix M), is used as a reference in the inspection of all kits and components.

1.7.2.8 Component items of first aid and/or survival kits that are deleted through normal supply action will be replaced or withdrawn from the sets and kits at the next normally scheduled inspection cycle. It is not necessary to recall the kits in use to effect replacement of deleted items unless otherwise notified through appropriate channels.

1.7.2.9 Recommend replacement of the kit if case or all components are unserviceable.

1.7.2.10 Identify possible abuse of the first aid kit or its contents and advise the appropriate unit commander and the MTF commander. If replacement of the first aid kit or its content is required, then it will be charged to the applicable USAF activity.

1.7.2.11 The previous First Aid Kit NSNs will remain suitable substitutes until the next scheduled inspection cycle in accordance with TO 00-35A-39, paragraph 1.7.2.8. It is not necessary to recall the kits in use to effect the changes. This is in effect for two years from the date of technical order publication.

## **1.8 INSPECTION BY OTHER THAN MEDICAL PERSONNEL.**

Using activities are responsible for periodically inspecting kits in their possession. This inspection is to insure that the kits have not been used, or subjected to an environment hostile to the kit. Kits which have been opened or whose outer appearance indicates possible damage to the contents will be turned in to the MMM for inspection and quality control. Inspections accomplished by using activities will be confined to an external examination of the container and its seals. The DD FORM 1574 or AFTO FORM 104 will be removed only by base medical supply personnel. Inspection actions (paragraph 1.7) will be coordinated with the MMM to insure a level workload is maintained in both the using activity and medical supply.

## **1.9 FUNDING FOR REPLACEMENT COMPONENTS.**

1.9.1 The MMM or his designated representative will replace missing components without charge to requesting USAF activities, including Air Reserve Components. Charges for replacement of component items will be made against RC/CC XX5864. The base medical Operations and Maintenance (O&M) program will budget for this cost. If replacement

requirements are excessive, the MMM will advise the base commander through the MTF commander and request an investigation or other appropriate action.

1.9.2 The cost of components used to repack kits for other than a USAF using activity (including Air Reserve Components) will be charged to the agency requesting the service.

1.9.3 If exterior cases and/or components are damaged, the requesting USAF activity will fund replacement of the new kit, new case, or affected components.

## **1.10 TURN IN OF EXCESS KITS.**

1.10.1 Unserviceable and excess components generated during the inspection by the MMM will be turned in and processed through the stock record account in accordance with AFI 41-209, Chapter 3 .

1.10.2 Medical first aid kits will be removed from aircraft prior to disposal or transfer of the aircraft to the Military Aircraft Storage and Disposition Center (MASDC). Using activities are responsible for returning these and all other excess or no longer authorized kits to the MMM. Reimbursement to using activities for kits turned in will not be given without the written approval of AFMSA/SGSLX, 1423 Sultan Drive, Ft. Detrick MD 21702-5006 for processing.

## **1.11 FURNISHING KITS TO AIR FORCE CONTRACTORS.**

1.11.1 Medical kits required for initial installation in new weapons systems will be funded by HQ AFMC. Funded requisitions for kits required by the terms of the Air Force contract will be submitted by the Air Force Plant Representative Office (AFPRO) through HQ AFMC/SGAR to AFMSA/SGSLX, 1423 Sultan Drive, Ft. Detrick MD 21702-5006, DSN 343-4638.

1.11.2 The director of materiel management at each AFMC ALC will compute requirements for kits being installed in aircraft or other major commodities for which they have modernization and/or maintenance responsibilities. These requirements will be funded by HQ USAF/SGHC. The contractor will submit their kit requirement to the director of materiel management at the applicable ALC, who in turn will submit unfunded requisitions to AFMSA/SGSLX, 1423 Sultan Drive, Ft. Detrick MD 21702-5006.

1.11.3 Contractors having unserviceable kits will request disposition instructions from AFMSA/SGSLX, 1423 Sultan Drive, Ft. Detrick MD 21702-5006.

## **1.12 KIT STANDARDIZATION OR MODIFICATION PROCEDURES.**

Recommendations to standardize additional medical kits or modify kits will be submitted as prescribed in AFMAN 23-110, Volume 5, chapter 16, paragraph 16-10, or using AFTO FORM 22, TECHNICAL ORDER SYSTEM PUBLICATION IMPROVEMENT REPORT AND REPLY. All such recommendations will be coordinated with both medical and life support authorities at each level of command.

## **1.13 NSN 6545-01-521-8511, FIRST AID KIT, GENERAL PURPOSE, RIGID CASE (FIGURE 1-2).**


1.13.1 PURPOSE. An assemblage of items designed to provide first aid material for use in life rafts, air-borne lifeboats, large aircraft and small water craft.

1.13.2 INSTALLATION. The USAF activity engaged in assembling survival kits for any of the above designed rafts, watercraft or aircraft is responsible for its installation.

1.13.3 MARKING. This kit will be marked in accordance with paragraph 1.4 of this TO.

1.13.4 INSPECTION. Using activities will return this kit to the MMM for the inspection required by paragraph 1.7. The MMM will replace all "P" coded items that have less than two years of shelf life remaining. Follow instructions in paragraph 1.7.2 for "P" coded items with less than two years shelf life remaining if they are needed to update the kit.

1.13.5 SEALING AND/OR TAGGING. Upon completion of the inspection and repacking, the MMM will affix a properly completed AFTO FORM 104 to the kit (Figure 1-2). The AFTO FORM will be affixed in a manner to preclude opening of the kit without tearing the form (Figure 1-2). Kits which have a torn AFTO FORM 104 will not be issued and will be returned to the MMM for reinspection.

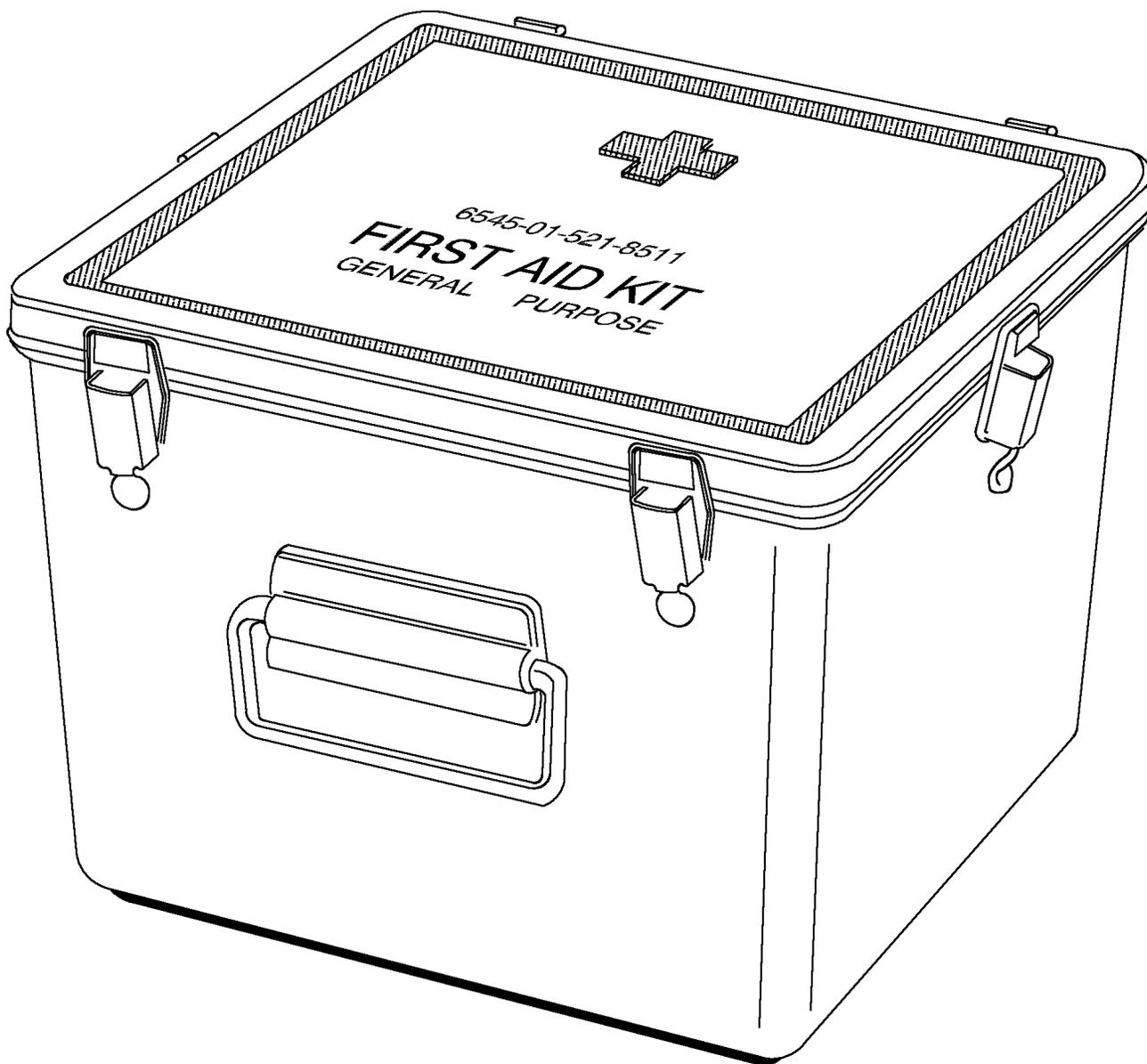

H9003177

Figure 1-2. First Aid Kit, General Purpose, Rigid Case, NSN 6545-01-521-8511 (Sheet 1 of 2)



H9003178

Figure 1-2. First Aid Kit, General Purpose, Rigid Case, NSN 6545-01-521-8511 (Sheet 2)

Table 1-2. NSN 6545-01-521-8511 - Components

NATIONAL STOCK NUMBER	NOTES	DESCRIPTION	QTY	UNIT OF ISSUE
6545-01-523-3668		CASE, MEDICAL INSTRUMENT AND SUPPLY SET, Size B, 9-1/2 in lg x 9 in wide x 7 in high.	1	EA
6515-01-494-1951		SPLINT, Universal	4	EA
6505-00-100-9985 (LP)	P	ASPIRIN TABLETS, USP, 0.324 GM, 100s.	1	BT
6505-00-926-2111 (LP)	P	MECLIZINE HYDROCHLORIDE TABLETS, USP, Chewable, 25 mg, 100s	1	BT
6505-01-121-2336	P	SUNSCREEN PREPARATION SPF 15, 4 oz bottle	2	BT
6508-01-436-0607		LIPSTICK, ANTICHAP, Cold or Hot Climate, SFP 15, 4.4 GM, Stick, 144/PG	3/144	PG
6510-01-460-0849		BANDAGE KIT, Elastic, 4 x 7	4	EA
6510-01-503-2117		BANDAGE, GAUZE, Compressed, 4.1 in x 4.5 yd, Sterile	8	RO
6510-00-201-1755		BANDAGE, MUSLIN, Compressed, Olive Drab, 37 x 37 x 52 inches, Triangular w/Safety Pins	4	EA
6510-00-202-0800		GAUZE, PETROLATUM, Stepback Accordion White, 18 in x 3 in, 12s	1	PG
6510-00-913-7909	P	BANDAGE, ADHESIVE, 75 x 3 inches, Flesh, 300s	30/300	BX
6510-01-060-1639	P	ADHESIVE TAPE, SURGICAL, 1.5 yd x 1 in, Surgical Rbbr Syn Coating Clear, 100	3/100	PG
6515-00-687-8052		AIRWAY, PHARYNGEAL, Berman Dsgn, Large Adult, 100MM, Lg Plastic Disposal	1/12	PG
6515-00-935-7138		SCISSORS, Bandage, 7.25 in, Lg Ang to Hdl, 1.50 in Cut, Lg Blunt PTS CRS	1	EA
6515-00-958-2232		AIRWAY, PHARYNGEAL, Berman Design, Child Sz, 80MM Length, 12s	1/12	BX
6515-01-364-8554		GLOVE, PATIENT EXAM/TREAT, Large Size, Pre-powdered, Snug Fitting Cuff, 100s	4/100	PG
6520-00-926-9041		BAG, Dental Prosthesis, Plastic, 100s	1/100	PG
6840-01-284-3982		INSECT REPELLENT, PERSONAL APPLICATION, for Mosquitoes, Ticks, Mites	1/12	PG
6850-00-985-7166	P	WATER PURIFICATION TABLET, IODINE, 8 mg, D-Day Item for DoD, 50s	1	BT
8315-00-787-8000 (GSA Item)		PIN, SAFETY, Brass, Sz-2, 1-1/2 in lg, 72s	1	PG
AFTO FORM 401		INSTRUCTION CARD, Artificial Respiration, Mouth- to-Mouth Resuscitation	1	EA
		TOTAL WEIGHT 17.282 TOTAL VOLUME 1.697		

## **1.14 NSN 6545-01-521-8530, SURVIVAL KIT, INDIVIDUAL (FIGURE 1-3).**

1.14.1 PURPOSE. This kit is designed to provide first aid items, toilet articles, and minimal nourishment to individual aircrew members stranded in isolated areas. Special instructions are as follows:

### **1.14.2 INSTALLATION.**

1.14.2.1 When used as a supplement to type I survival kit assemblages, the USAF activity engaged in assembling such kits, will install this item in its designated location.

1.14.2.2 When issued as an item of individual equipment its issue by the individual equipment activity will be recorded on AF FORM 538, PERSONAL CLOTHING AND EQUIPMENT RECORD, or on AF FORM 1297, TEMPORARY ISSUE RECEIPT. Personal equipment activities will establish a method to insure the item is returned every 36 months for inspection and repacking. The kit, when issued as an individual equipment item, will be carried securely in pockets of the outer garment in such a manner that it will not be cumbersome or dangerous to the individual during bail out or ditching procedures.

1.14.3 INSPECTION AND REPLACEMENT OF COMPONENTS. This kit will be inspected and quality assurance actions performed by the MMM in accordance with paragraph 1.7 of this TO. Using activities will establish procedures to insure all kits are returned to the MMM for this inspection. All items identified with note code "P" will be replaced by the MMM if they have less than two years of shelf life remaining, unless items are not available; follow instructions in paragraph 1.7.2.2 for "P" coded items with less than two years shelf life remaining if they are needed to update the kit. Subsistence items will be replaced by items containing a later date of manufacture during each inspection of the kits by the MMM.

1.14.4 SOURCE OF SUPPLY AND SPECIFICATIONS FOR NONSTANDARD ITEMS. Items coded LP in the component list may be procured from the manufacturer of the item or a local distributor. Before purchasing from a local source, check to make sure the item has not been recently depot stocked. Be aware that manufacturers and distributors have minimum order quantities.

1.14.5 REPACKING OF KIT. All components of this kit are to be enclosed in a waterproof enclosure. Replacement items which are not sealed in a primary plastic or aluminum enclosure will be sealed in polyethylene bags by the MMM. NSN 6850-00-985-7166, Water Purification Tablets, Iodine 8 mg, 50's that are sealed with paraffin wax do not require additional sealing. If upon visual inspection, the wax seal is broken, the bottle should be replaced.

## TO 00-35A-39

1.14.5.1 The polyethylene bags specified herein shall be made from clear extruded stainless LAYFLAT TUBING. Polyethylene shall be 100 percent virgin material having an average wall thickness of 0.003 inch. Minimum thickness at any point shall not be less than 0.00225 inch. Both seam and closure shall be heat sealed to prevent moisture from entering or to prevent escape of fluid contents in the event of leakage; excess air shall have been excluded from the sealed bag so that bag conforms to the shape of the component therein. To insure a minimum of excess material, the width of commercially available tubing shall be of a dimension which shall closely fit the individual component to be enclosed. The length of the tubing between seals shall be the minimum to contain the component. Excess tubing beyond the seals shall be cut off. A properly sealed bag can be submerged to a depth of one foot in tap water at room temperature for 15 minutes without leaking.

1.14.5.2 Attach tape handles to kit by using tape pressure sensitive adhesive, low or medium tensile strength, approximately 3/4 inch wide. Suggested stock numbers for tape are 7510-00-290-8035, 7510-00-802-8311, 7510-00-290-8036 or any equivalent. Handles are added to provide a carrying handle for ease in opening, and for attachment of kit to survival vest mesh net. These handles will be added by placing a piece of tape lengthwise to the bottom of the case and another attached to the top cover at right angles, each length extending approximately edge to edge, sufficient to support the kit. Leave a loop or handle at both top and bottom of kit, with 5-6 inches of slack for ease of grasping with glove hand. [Figure 1-3](#) provides guidance for location

1.14.6 MARKING. This kit does not require a red cross symbol as it is an individual item of equipment.

1.14.7 SEALING AND/OR TAGGING. Upon completion of the inspection and repacking, the MMM will affix a properly completed AFTO FORM 104 to the kit. The AFTO FORM will be affixed in a manner to preclude opening of the kit without tearing of the form. Kits which have a torn AFTO FORM 104 will not be issued. All such kits will be returned to the MMM for reinspection.

Table 1-3. NSN 6545-01-521-8530 - Components

ITEM NO.	NATIONAL STOCK NUMBER	NOTES	DESCRIPTION	QTY	UNIT OF ISSUE
	6545-01-189-7094		CASE, INDIVIDUAL SURVIVAL KIT, Plastic, Polyethylene Opaque	1	EA
13	4220-01-181-3154		FISHING KIT EMERGENCY	1	EA
3	6510-00-200-3080		COMPRESS/BANDAGE, Sterile, Camouflaged, Compress, 4 x 4 in Pad Fixed, 84 in Bandage	1	EA
12	6510-00-913-7909	P	BANDAGE, ADHESIVE,.75 x 3 inches, Flesh, 300s	6/300	BX
	6510-01-060-1639	P	ADHESIVE TAPE, Surgical, 1.5 yd x 1 in Surg Rbbr Syn Coating, Clear, 100	1/100	PG
	6515-00-296-2529		SAW, Flexible, Steel Wire, Cadmium Plated, 16 Teeth/Inch, 16 in lg	1	EA
	6515-01-337-3689		GLOVE, Patient Exam/Treatment Mens/Womens Medium Prepowdered Rubber, 100s	4/100	PG
7	6850-00-985-7166	P	WATER PURIFICATION TABLETS, IODINE, 8 mg, D-Day Item for DoD, 50s	1	BT
	7610-01-301-5259 (LP)		LIST, MEDICAL INSTRUCTIONS -CONTENTS u/w 6545-00-139-3671 Survival Kit	1	EA
2	8520-00-782-3554		HAND CLEANER	3/2000	BX
10	8925-01-008-0960 (LP)		CANDY, CHOCOLATE	1	BG
4	8935-00-082-5667 (LP)		SOUP AND GRAVY BASE, Powder Form, Chicken Flavor, 7 grams	2/1000	PG
5	9535-01-201-7014 (LP)		FOIL w/SURVIVAL KIT, NSN 6545-01-521-8530	1	EA
9	9920-01-191-3434 (LP)		MATCHES, SAFETY, Humidity Resistant, 2 x 1.50 x 0.25 in Water Proof Bag, 20s	1	PG
			TOTAL WEIGHT 1.613 TOTAL VOLUME 0.728		
NSN 4220-01-181-3154, Fishing Kit Emergency (3 Components) Available from Opportunity Resources, POC Candace Lewis, 406-721-2930 and Specialty Products, Inc., 6626 Easton Rd, Pipersville, PA 18947, POC Fred Prozzillo at 215-766-7777.					
NSN 8935-00-082-5667, Soup and Gravy Base, Instant, Chicken Flavored, 8 gram envelopes 1000s, Managed by DSC-P Subsistence Opportunity Resources, Inc., POC Candace Lewis, 406-721-2930, and TCOMUS Co., POC Bill Chervenak, 931-520-0005.					

**1.15 NSN 6545-01-521-8528, SURVIVAL KIT, INDIVIDUAL, TROPICAL, TACTICAL AIRCREW (FIGURE 1-4).**

1.15.1 PURPOSE. This kit is specifically designed to provide a one week supply of first aid and survival supplies to stranded aircrew members during escape and evasion situations encountered under tropical conditions.

1.15.2 INSTALLATION. This kit will be issued in accordance with paragraph 1.14.2 of this TO.

1.15.3 INSPECTION. Using activities will return this kit to the MMM for inspection required by paragraph 1.7. The MMM will replace all "P" coded items that have less than two years of shelf life remaining unless items are not available. Follow instructions in paragraph 1.7.2 for "P" coded items with less than two years of shelf life remaining if they are needed to update the kit.

1.15.4 REPACKING.


1.15.4.1 Components of this kit are installed as shown in Figure 1-4. Aluminum foil strip items are to be folded to fit the designated places. Rubber bands are to be used to keep the strips folded. Items which are not encased in aluminum foil will be encased in a polyethylene bag as prescribed in paragraph 1.14.5.1.

1.15.4.2 The inner surface of the bottom piece of the plastic case is coated with a layer of non-curing, pressure sensitive adhesive. This adhesive is provided to secure positioning of the components in their respective locations while the case is opened or inverted.

1.15.4.3 Items coded LP in the component listing may be procured from the manufacturer of the item or a local distributor. Before purchasing from a local source, check to make sure the items has not been recently depot stocked. Be aware that manufacturers and distributors have minimum order quantities.

1.15.5 MARKING. This kit does not require application of a red cross symbol.

1.15.6 SEALING AND/OR TAGGING. After completion of the inspection and repacking, the kit will be tagged with two AFTO FORMs 104 - one on each side of the kit, see Figure 1-1. The AFTO FORMs 104 will be affixed in a manner to preclude opening the kit without tearing the forms. Kits which have a torn AFTO FORM 104 will not be issued by life support or using activities. All such kits will be returned to the MMM for reinspection.

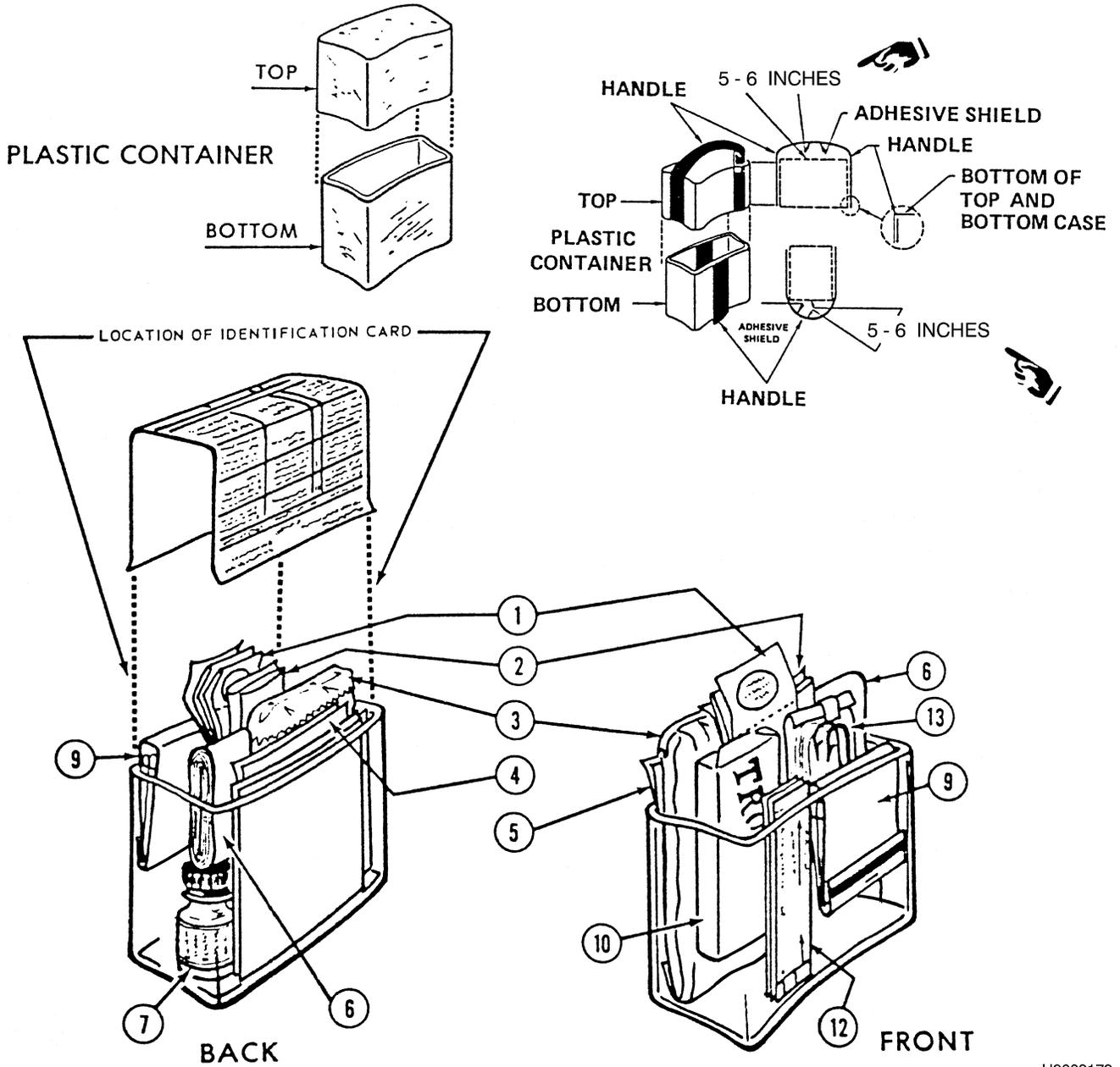

H9003179

Figure 1-3. Kit Assembly

Table 1-4. NSN 6545-01-521-8528 - Components

ITEM NO.	NATIONAL STOCK NUMBER	NOTES	DESCRIPTION	QTY	UNIT OF ISSUE
	6545-01-189-7095		CASE, INDIVIDUAL, Survival Kit, Plastic	1	EA
	4220-01-181-3154		FISHING KIT EMERGENCY	1	EA
12	6510-00-913-7906		BANDAGE, GAUZE, ELASTIC, Two-Ply, 2 inches by 5 yards, 50s	1/50	BX
13	6510-00-913-7909	P	BANDAGE, ADHESIVE,.75 by 3 inches, Flesh, 300s	12/300	BX
14	6510-01-060-1639	P	ADHESIVE TAPE, SURGICAL, 1.5 yd x 1 in, Surg Rbbr, Syn Coating, Clear, 100s	1/100	PG
9	6515-00-296-2529		SAW, Flexible, Steel Wire Cadmium Plated, 16 teeth/inch, 16 inch long	1	EA
11	6515-01-363-1212		RAZOR, SURGICAL PREPARATION, Small, Folding Handle, Str Type, Sgl Blade, 100s	1/100	PG
	6515-01-337-3689		GLOVE, PATIENT EXAM/TREATMENT, mens/womens, Medium, Pre-powdered, rubber, 100s	4 gloves or 2 pair	PG
10	6605-01-326-1654		COMPASS	1	EA
21	6695-00-118-2918 (LP)		BAG, FOOD SAMPLE, Polyethylene, Disposable, 9 in x 4.5 in, 18 fl oz capacity, 500s	5/500	PG
17	6840-00-142-8965		INSECT REPELLENT, Personal, 1 oz, 12s	1/12	BX
5	6850-00-985-7166	P	WATER PURIFICATION TABLET, IODINE, 8 mg, D-Day Item for DoD, 50s	1	BT
19	7610-01-301-5257		LIST MEDICAL INSTRUCTIONS - CONTENTS, Used with 6545-01-521-8528	1	EA
16	8415-01-192-2357		INSECT NET MITTENS SET, Head and Hand Covering, Nylon, Black	1	EA
20	9920-01-191-3434 (LP)		MATCHES, Safety, Humidity Resistant, Waterproof Bag	1	PG
			TOTAL WEIGHT 1.069 TOTAL VOLUME 0.341		

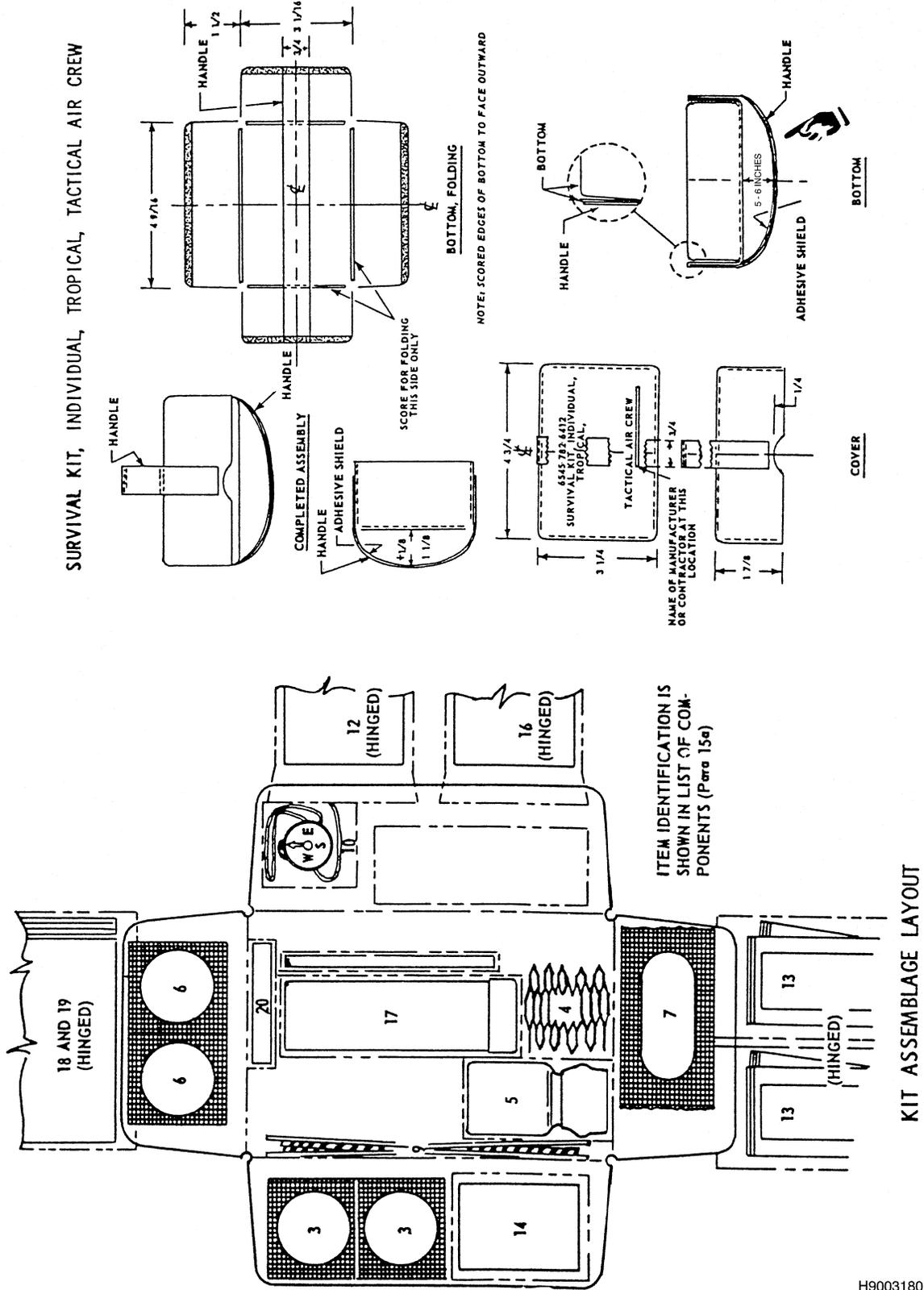


Figure 1-4. Survival Kit, Individual, Tropical, Tactical Aircrew, NSN 6545-01-521-8528

H9003180

**1.16 NSN 6545-01-521-8519, FIRST AID KIT GENERAL PURPOSE, FLEXIBLE CASE, PANEL MOUNTED (FIGURE 1-5).**

1.16.1 PURPOSE. This kit is an assemblage of items designed to provide first aid supplies to personnel on board an aircraft or watercraft.

**NOTE**

This item is not suitable for being carried on the person during bail out procedures.

1.16.2 INSTALLATION. The USAF activity responsible for equipping waterborne craft or aircraft with authorized first aid and/or survival equipment will install this kit in accordance with applicable TOs, parts lists or drawings. Parts or material required to provide brackets or holders for installation of the kit will be obtained from Air Force stock or locally manufactured. The maintenance or repair activity will install the mountings.

1.16.3 INSPECTION. This kit will be inspected in accordance with paragraph 1.7.

1.16.4 REPACKING. The contents of this kit will be installed in the sections indicated in the component list.

1.16.5 TAGGING. Prior to issue of this kit, it will be tagged by medical material personnel as follows:

1.16.5.1 Close the zipper completely with the side fastener tab in the down and locked position.

1.16.5.2 Using seal, NSN 5340-01-004-9387, pass through eyelet of DD FORM 1574 and through the hole of the zipper slide fastener, then lock.


1.16.5.3 If the operation is correctly performed you will be unable to open the zippered section of the kit without breaking the seal. This method provides positive external visual indicator of kit serviceability.

1.16.6 MARKING. The outer container of the kit will be marked in accordance with paragraph 1.4.

Table 1-5. NSN 6545-01-521-8519 - Components

NATIONAL STOCK NUMBER	NOTES	DESCRIPTION	QTY	UNIT OF ISSUE
6545-01-523-3645		CASE, MEDICAL, INSTRUMENT AND SUPPLY SET, Polyamide Nylon, Olive	1	EA
<b>IN UPPER POCKET</b>				
6510-00-201-1755		BANDAGE, MUSLIN, Compressed, Olive Drab, 37 x 37 x 52 in, Triang w/Safety Pins	2	EA
6510-01-112-6414		GAUZE, PETROLATUM, 3 x 36 inches, Stepback, Accordion White, 12s	3/12	PG
6515-01-363-1212		RAZOR SURG PREPARATION, Small Folding HDL STR Type SGL Blade, 100s	1	PG
6515-01-364-8554		GLOVE, Patient Exam/Treatment, Large Size, Pre-powdered, Snug Fitting Cuff, 100s	4/100	PG
6520-00-926-9041		BAG, Dental Prosthesis, Plastic, 100s	1/100	PG
6910-01-515-4278		SHIELD, Face, Cardioulmonary Resuscitation	1	EA
<b>IN LOWER POCKET</b>				
6510-01-460-0849		BANDAGE KIT, Elastic, 4 x 7	2	EA
6510-01-503-2117		BANDAGE, GAUZE, Compressed, 4.1 in x 4.5 yd, Sterile	2	RO
6510-00-913-7909	P	BANDAGE, ADHESIVE,,75 x 3 inches, Flesh, 300s	18/300	BX
6510-01-060-1639	P	ADHESIVE TAPE, Surgical, 1 inch x 1-1/2 yards, Surg Rbbr, Syn Coating, Clear, 100s	3/100	PG
AFTO FORM 401		INSTRUCTION CARD, Artificial Respiration (Mouth-to- Mouth Resuscitation)	1	EA
		TOTAL WEIGHT 1.919 TOTAL VOLUME 0.115		



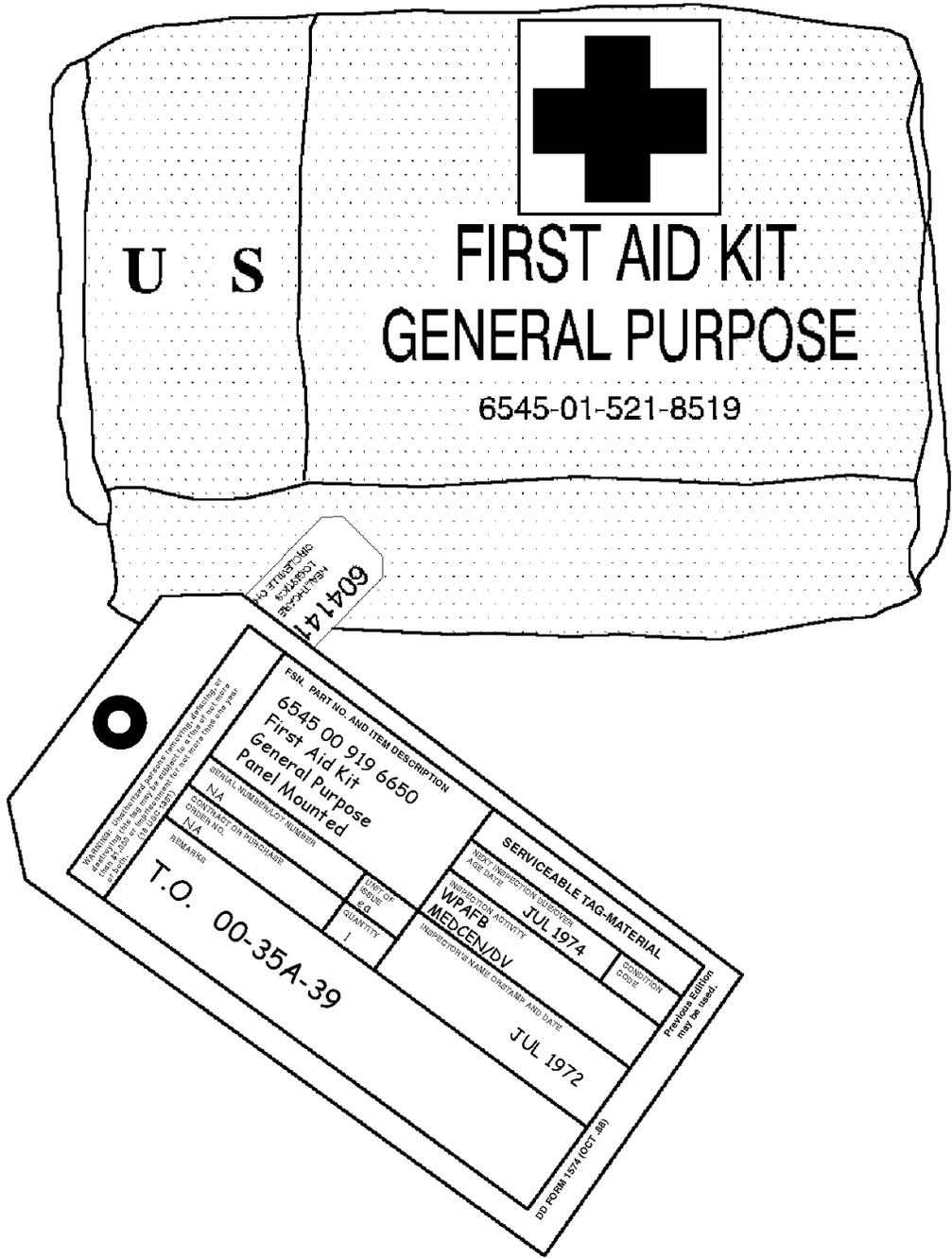

H9003183

Figure 1-5. First Aid Kit, General Purpose, Flexible Case, Panel Mounted, NSN 6545-01-521-8519 (Sheet 1 of 2)



**1.17 NSN 6545-01-521-8515, FIRST AID KIT GENERAL PURPOSE, RIGID CASE (FIGURE 1-6).**

1.17.1 PURPOSE. This kit is an assemblage designed to provide a source of emergency first aid items for vehicle operators or passengers, and to satisfy requirements specified for use in AFOSH Standard 127-50, Ground Communications - Electronics (C-2) Systems, paragraph 4-f(5)(a)4.

1.17.2 INSTALLATION. This kit will be installed by the local maintenance people in a manner that will assure that its container will be firmly retained by locally provided brackets but which will allow the kit to be readily removed without tools for inspection or an emergency.


1.17.3 INSPECTION. This kit will be returned every 36 months to the MMM for inspection as outlined in paragraph 1.7.

1.17.4 MARKING. This kit will be marked as outlined in paragraph 1.4.

1.17.5 SEALING AND/OR TAGGING. After completion of the inspection and repacking, the kit will be tagged with AFTO FORM 104 (Figure 1-1). The AFTO FORM 104 will be affixed in a manner to preclude opening the kit without tearing the form. Kits which have a torn AFTO FORM 104 will not be issued by life support or using activities. All such kits will be returned to MMM for reinspection.

Table 1-6. NSN 6545-01-521-8515 - Components

NATIONAL STOCK NUMBER	NOTES	DESCRIPTION	QTY	UNIT OF ISSUE
6545-01-523-3674 (LP)		CASE, MEDICAL INSTRUMENT AND SUPPLY SET, 7.5 x 4.5 x 2.75 in, Olive Drab	1	EA
6510-01-460-0849		BANDAGE KIT, Elastic, 4 x 7	2	EA
6510-01-503-2117		BANDAGE GAUZE, Compressed, 4.1 in x 4.5 yd, Sterile	2	RO
6510-00-201-1755		BANDAGE, MUSLIN, Compressed, Olive Drab, 37 x 37 x 52 in, Triang w/Safety Pins	2	EA
6510-00-913-7909	P	BANDAGE, ADHESIVE, .75 x 3 inches, Flesh, 300s	18/300	BX
6510-00-935-5820		BANDAGE, ELASTIC, Flesh, Rolled, Nonster, 4.5 yd x 2 in pressure wash, 12s	1/12	PG
6510-01-060-1639	P	ADHESIVE TAPE, Surgical, 1.5 yd x 1 in Surg Rbbr, Syn Coating, Clear, 100	3/100	PG
6510-01-112-6414		GAUZE, PETROLATUM, 3 x 36 inches, Stepback Accordion White, 12s	3/12	PG
6515-01-363-1212		RAZOR SURG PREPARATION, Small Folding HDL STR Type SGL Blade 100s	1	PG
6515-01-364-8554		GLOVE, Patient Exam/Treat, Large Size, Pre-powdered, Snug Fitting Cuff, 100s	4/100	PG
6520-00-926-9041		BAG, Dental Prosthesis, Plastic, 100s	1/100	PG
AFTO FORM 401		INSTRUCTION CARD, Artificial Respiration (Mouth-to- Mouth Resuscitation)	1	EA
		TOTAL WEIGHT	4.593	
		TOTAL VOLUME	0.247	

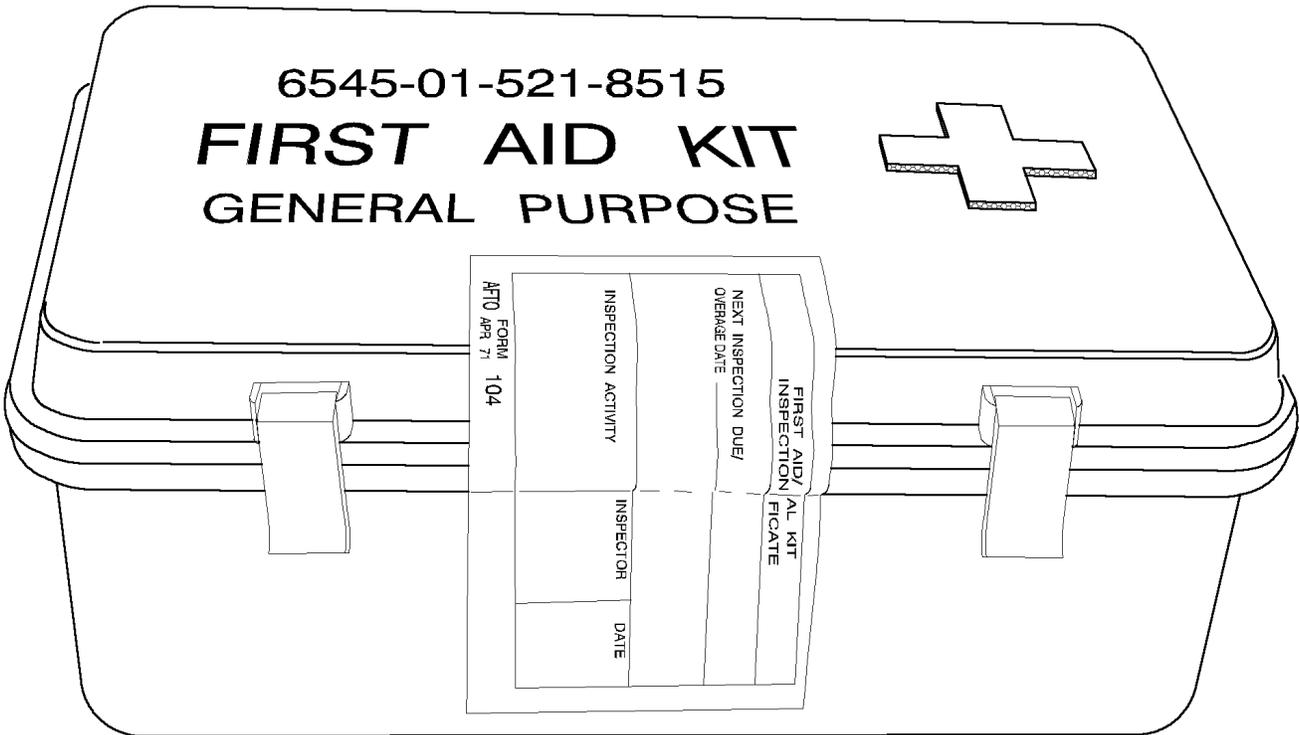

H9003173

Figure 1-6. First Aid Kit, General Purpose, Rigid Case, NSN 6545-01-521-8515 (Sheet 1 of 2)



**1.18 NSN 6545-01-521-8514, FIRST AID SET, SHELTER.**

1.18.1 PURPOSE. This kit provides an assembly of first aid shelter supplies for field use.

1.18.2 INSTALLATION.

1.18.2.1 Materiel requirements for this purpose will consist of one set, first aid, shelter, NSN 6545-01-521-8514 and six each litter, rigid pole, for each 100 programmed military personnel.

1.18.2.2 Items prepositioned under this program will be maintained by the supporting stock record account. The major commands will develop arrangements to insure materiel availability to satellite and collocated operating base personnel.

1.18.2.3 This materiel will be issued to supported units only under predetermined defense conditions as specified by the major command. Accordingly, local plans will include procedures for rapid issue of this materiel upon direction of the major command.

1.18.3 INSPECTION. This item will be inspected every 36 months as prescribed in paragraph 1.7.

1.18.4 MARKING. This kit will be marked in accordance with paragraph 1.4.

1.18.5 SEALING AND/OR TAGGING. Upon completion of the inspection and repacking the MMM will affix a properly completed AFTO FORM 104 to the kit. The AFTO FORM will be affixed in a manner to preclude opening of the kit without tearing of the form. Kits which have torn AFTO FORM 104 will not be issued. All such kits will be returned to the MMM for reinspection.

Table 1-7. NSN 6545-01-521-8514 - Components

NATIONAL STOCK NUMBER	NOTES	DESCRIPTION	QTY	UNIT OF ISSUE
6510-00-111-0708		PAD, Non-Adherent, 3.875/4.125 x 2.875/3.125 in, White Sterile, I.S., 100s	1	PG
6510-01-460-0849		BANDAGE KIT, Elastic, 4 x 7	12	EA
6510-01-503-2117		BANDAGE, GAUZE, COMPRESSED, 4.1 in x 4.5 yd, Sterile	12	RO
6510-00-201-1755		BANDAGE, MUSLIN, Compressed, Olive Drab, 37 x 37 x 52 inches, w/Safety Pins	10	EA
6510-00-201-7430		DRESSING, FIRST AID, Field, Camouflaged, 7.75-8.25 in Lg, 7.25-7.75 Wd	10	EA
6510-00-203-5500	P	ADHESIVE TAPE, Surgical, 3 in x 5 yd	1	RO
6510-00-597-7469	P	BANDAGE, ADHESIVE, .75 x 3 inches, Flesh/Clear Ster Dress, Affixed to Plas ADH, 100s	1	PG
6510-00-782-2698		SPONGE, SURGICAL, Gauze, ABS White, 4 x 4 in, 8 ply, U/ F Injuries or Surgery, 200	1	PG
6510-00-935-5822		BANDAGE, Elastic, Flesh, Rolled, 4.5 yd x 4 in, washable pressure, 12s	1	PG
6515-01-521-7976		C-A-T Tourniquet	2	EA
6515-01-494-1951		SPLINT, Universal	12	EA
6515-00-935-7138		SCISSORS, Bandage, 7.25 in, Lg Ang to Hdl, 1.50 in cut lg blunt PTS CRS	5	EA
6515-01-364-8554		GLOVE, Patient Exam/Treat Large Sz, Prepwdr, Snug Fitting Cuff, 100s	10/ 100	PG
6520-00-926-9041		BAG, Dental Prosthesis, Plastic, 100s	1/100	PG
6530-01-380-7309		LITTER, Folding, Rigid Pole	6	EA
6530-00-784-4205		STRAP, Webbing, Patient Securing, Buckle with Locking Device and Spring	12	EA
6545-01-523-3679		CHEST, Misc Field, No. 5, 30 in lg x 18 in wide x 16 in depth	1	EA
7210-00-715-7985		BLANKET, Bed, Wool, Olive Green, 90 x 66 in	6	EA
8315-00-787-8000 (GSA Item)		PIN, Safety, Brass, Size-2, 1-1/2 in lg, 72s	24/72	PG
AFTO FORM 401		INSTRUCTION CARD, Artificial Respiration (Mouth-to-Mouth Resuscitation)	1	EA
		TOTAL WEIGHT 115.021 TOTAL VOLUME 12.863		

**1.19 NSN 6545-01-521-8502, FIRST AID KIT, INDIVIDUAL (FIGURE 1-7).**

Packed in case 6545-01-094-6142.

**1.19.1 PURPOSE.**

1.19.1.1 The kit provides first aid items for aircrew members to treat injuries sustained while separated from their aircraft during emergencies. Preferred kit has been enlarged for sufficient space to hold all components.

1.19.1.2 Kit is a component of standard mobility bag.

1.19.1.3 This kit is to be included as the individual first aid kit for mobility bags. NSN 6545-01-521-8502 is considered complete for the basic Self Aid and Buddy Care (SABC) concept.

**1.19.2 INSTALLATION.**

1.19.2.1 When used in various major survival kits, or standard mobility bags, the USAF activity responsible for assembling the major items will install this kit.

1.19.2.2 When issued as an item of individual equipment on AF FORM 538 or 1297, the kit will be carried in an outer pocket or on a vertical strap of the field clothing except during bail out procedures.

1.19.3 INSPECTION. Using activities will return this kit to the MMM for inspection required by paragraph 1.7. The MMM will replace all "P" coded items with less than two years of shelf life remaining unless items are not available. Follow instructions in paragraph 1.7.2 for "P" coded items with less than two years shelf life remaining if they are needed to update the kit.

1.19.4 MARKING. No special markings are required.

1.19.5 PACKING AND REPACKING. Components of this kit are packed into the plastic insert (6545-01-094-6136) except for one Bandage Kit, Elastic (6510-01-460-0849). The plastic insert is then placed inside the Nylon case (6545-01-094-6142). The remaining Bandage Kit, Elastic will be inserted behind the plastic insert inside the Nylon case.


1.19.6 SEALING AND/OR TAGGING. After completion of the inspection and repacking, AFTO FORM 104 will be used to tag the kit. The AFTO FORM 104 will be affixed NSN 6545-01-094-6136 insert, first aid kit case, plastic, olive drab, in a manner to preclude opening the kit without tearing the form. Kits which have a torn AFTO FORM 104 will not be issued by life support or using activities. All such kits will be returned to the MMM for reinspection.

Table 1-8. NSN 6545-01-521-8502 - Components

NATIONAL STOCK NUMBER	NOTES	DESCRIPTION	QTY	UNIT OF ISSUE
6545-01-523-3689		CASE, MEDICAL INSTRUMENT SUPPLY SET, 4.75 in x 2 3/8 in x 5.50 in	1	EA
6545-01-094-6136		INSERT, FIRST AID KIT CASE, Plastic OD, 4.750 x 2.375 x 5.500 in box	1	EA
6508-01-436-0607		LIPSTICK, ANTICHAP, Cold or Hot Climate, SPF 15, 4.4 GM, Stick	1/144	PG
6510-01-460-0849		BANDAGE KIT, Elastic, 4 x 7	2	EA
6510-01-503-2117		BANDAGE, GAUZE, Compressed, 4.1 in x 4.5 yd, Sterile	1	RO
6510-00-201-1755		BANDAGE, MUSLIN, Compressed, Olive Drab, 37 x 37 x 52 inches, Triangular w/Safety Pins	1	EA
6510-00-913-7909	P	BANDAGE, ADHESIVE,.75 x 3 inches, Flesh, 300s	10/300	BX
6850-00-985-7166	P	WATER PURIFICATION TABLET, IODINE, 8 MG, D-Day Item for DoD, 50s	1	BT
AFTO FORM 401		INSTRUCTION CARD, Artificial Respiration, Mouth-to-Mouth Resuscitation	1	EA
		TOTAL WEIGHT 1.285		
		TOTAL VOLUME 0.100		



Figure 1-7. First Aid Kit, Individual, NSN 6545-01-521-8502 (Sheet 1 of 2)

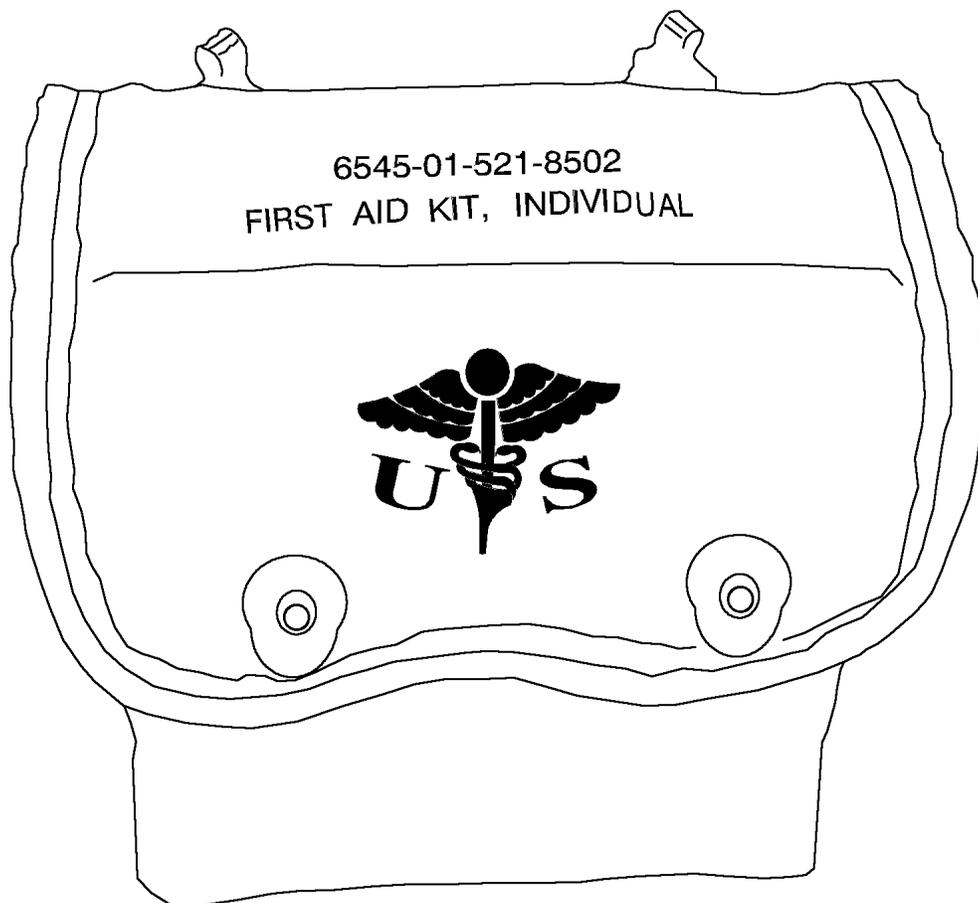

H9003176

Figure 1-7. First Aid Kit, Individual, NSN 6545-01-521-8502 (Sheet 2)

## **1.20 NSN 6545-01-521-8522, SURVIVAL KIT, INDIVIDUAL, TROPICAL, TACTICAL AIRCREW, IN A PLASTIC BAG.**

1.20.1 **PURPOSE.** This kit is specifically designed to provide a one week supply of first aid and survival supplies to stranded aircrew members during escape and evasion situations encountered under tropical conditions. This soft pack kit is mandatory for use with aircrew chemical defense equipment (ACDE) in accordance with TO 14S1-3-51. When ACDE is not required for use by aircrew members, this soft pack kit or NSN 6545-01-521-8528 can be installed in the SRU-21P survival vest, 12 pockets.

1.20.2 **INSTALLATION.** This kit will be issued in accordance with paragraph 1.14.2 of this TO.

1.20.3 **INSPECTION.** Using activities will return this kit to the MMM for inspection required by paragraph 1.7. The MMM will replace all "P" coded items that have less than two years of shelf life remaining unless items are not available. Follow instructions in paragraph 1.7.2 for "P" coded items with less than two years of shelf life remaining if they are needed to update the kit.

### **1.20.4 PACKING AND REPACKING.**

1.20.4.1 Components of this kit are to be packed as flat as possible to keep the kit pliable for the survival vest. Aluminum foil strips are to be tucked under to prevent rupture of the inner plastic bag. Rubber bands are to be used to keep the strips folded. Items not encased in aluminum foil will be encased in a polyethylene bag as prescribed in paragraph 1.14.5.1. The completed kit packed in the first ziploc bag NSN 8105-00-837-7754 will be inserted (zippered end first) into the additional or second ziploc bag. This will provide more durability for the kit while maintaining the size and flexibility of the current configuration. The completed/packed kit will not exceed 6 inches length.

1.20.4.2 Items coded LP in the component listing may be procured from the manufacturer of the item or a local distributor. Before purchasing from a local source, check to make sure the item has not been recently depot stocked. Be aware that manufacturers and distributors have minimum order quantities.

1.20.5 **MARKING.** This kit does not require application of a red cross symbol.

1.20.6 **SEALING AND/OR TAGGING.** After completion of the inspection and repacking, the kit will be tagged with AFTO FORM 104 across the opening of the inner plastic bag. The AFTO FORM 104 will be affixed in a manner to preclude opening of the inner bag without tearing the form. Kits which have a torn AFTO FORM 104 will not be issued by life support or using activities. All such kits will be returned to the MMM for reinspection.

Table 1-9. NSN 6545-01-521-8522 - Components

NATIONAL STOCK NUMBER	NOTES	DESCRIPTION	QTY	UNIT OF ISSUE
8105-01-521-0403		BAG, PLASTIC, Interlock, Seal Closure, Fastening, 6 in x 6 in/4 mil	1/1000	PG
4220-01-181-3154		FISHING KIT, EMERGENCY	1	EA
6510-00-054-7255		SKIN CLOSURE, ADHESIVE, SURGICAL, POROUS,.25 x 4 in, 500s	10/500	PG
6510-00-913-7906		BANDAGE, GAUZE, ELASTIC, 2-Ply, 2 in x 5 yd, 50s	1/50	BX
6510-00-913-7909	P	BANDAGE, ADHESIVE,.75 x 3 inches, Flesh, 300s	12/300	BX
6510-01-483-3333	P	TOWELETTE Benzalkonium, CHL Impregnated, 8 x 5 in, Alcohol-free, 100s	5/100	PG
6515-00-296-2529		SAW, Flexible Steel, Wire Cadmium Plated, 16 Teeth/in, 16 in long	1	EA
6515-01-363-1212		RAZOR, SURGICAL, PREPARATION, Small Folding Hdl, Str Type, Sgl Blade, 100x	1/100	PG
6695-00-118-2918 (LP)		BAG, FOOD SAMPLE, Polyethylene Disp, 9 in x 4.5 in, 18 fl oz capacity, 500s	3/500	PG
6850-00-985-7166	P	WATER PURIFICATION TABLET, IODINE, 8 mg, D-Day Item for DoD, 50s	1	BT
7610-01-301-5255 (LP)		LIST MEDICAL INSTRUCTIONS - Contents used with 6545-01-521-8522	1	EA
*8415-01-192-2357		INSECT NET MITTENS SET, Head and Hand Covering, Nylon, Black	1	EA
8520-00-551-0375 (GSA Item)		SOAP, TOILET, Nonmedicated Cake I.W. 0.75 oz, 1000s	1/1000	MX
9920-01-191-3434 (LP)		MATCHES, Safety, Humidity Resistant, 2 x 1.50 x 0.25 in, Waterproof Bag, 20s	1	PG
		TOTAL WEIGHT 1.096 TOTAL VOLUME 0.041		

**1.21 COMPONENT CROSS REFERENCE TO FIRST AID KITS.**

This table is designed to cross reference a specific component to the kit NSNs it is in. The cross reference is in NSN component sequence.

**Table 1-10. Component Cross Reference to First Aid Kits**

COMPONENT NSN	KIT NSN
4220-01-181-3154	6545-01-521-8522
	6545-01-521-8528
	6545-01-521-8530
6505-00-100-9985	6545-01-521-8511
6505-00-926-2111	6545-01-521-8511
6505-01-121-2336	6545-01-521-8511
6508 01-436-0607	6545-01-521-8502
	6545-01-521-8511
6510-00-054-7255	6545-01-521-8522
6510-00-111-0708	6545-01-521-8514
6510-00-200-3080	6545-01-521-8530
6510-00-201-1755	6545-01-521-8502
	6545-01-521-8511
	6545-01-521-8514
	6545-01-521-8519
6510-00-201-7430	6545-01-521-8514
6510-00-202-0800	6545-01-521-8511
6510-00-203-5500	6545-01-521-8514
6510-00-597-7469	6545-01-521-8514
6510-00-782-2698	6545-01-521-8514
6510-00-913-7906	6545-01-521-8522
	6545-01-521-8528
6510-00-913-7909	6545-01-521-8502
	6545-01-521-8511
	6545-01-521-8522
	6545-01-521-8515
	6545-01-521-8519
	6545-01-521-8528
	6545-01-521-8530
6510-00-935-5820	6545-01-521-8515
6510-00-935-5822	6545-01-521-8514
6510-01-060-1639	6545-01-521-8511
	6545-01-521-8515
	6545-01-521-8519
	6545-01-521-8528
	6545-01-521-8530
6510-01-112-6414	6545-01-521-8515
	6545-01-521-8519
6510-01-460-0849	6545-01-521-8502

Table 1-10. Component Cross Reference to First Aid Kits - Continued

COMPONENT NSN	KIT NSN
	6545-01-521-8511
	6545-01-521-8514
	6545-01-521-8515
	6545-01-521-8519
6510-01-483-3333	6545-01-521-8522
6510-01-503-2117	6545-01-521-8502
	6545-01-521-8511
	6545-01-521-8514
	6545-01-521-8515
	6545-01-521-8519
6511-00-201-1755	6545-01-521-8515
6515-00-296-2529	6545-01-521-8522
	6545-01-521-8528
	6545-01-521-8530
6515-00-687-8052	6545-01-521-8511
6515-00-935-7138	6545-01-521-8511
	6545-01-521-8514
6515-00-958-2232	6545-01-521-8511
6515-01-337-3689	6545-01-521-8528
	6545-01-521-8530
6515-01-363-1212	6545-01-521-8522
	6545-01-521-8515
	6545-01-521-8519
	6545-01-521-8528
6515-01-364-8554	6545-01-521-8511
	6545-01-521-8514
	6545-01-521-8515
	6545-01-521-8519
6515-01-494-1951	6545-01-521-8511
	6545-01-521-8514
6515-01-521-7976	6545-01-521-8514
6520-00-926-9041	6545-01-521-8511
	6545-01-521-8514
	6545-01-521-8515
	6545-02-521-8519
6530-00-784-4205	6545-01-521-8514
6530-01-380-7309	6545-01-521-8514
6545-00-113-3722	6545-01-521-8515
6545-00-131-6992	6545-01-521-8511
6545-00-912-9860	6545-01-521-8519
6545-00-914-3500	6545-01-521-8514
6545-01-094-6136	6545-01-521-8502
6545-01-094-6142	6545-01-521-8502
6545-01-189-7094	6545-01-521-8530

Table 1-10. Component Cross Reference to First Aid Kits - Continued

COMPONENT NSN	KIT NSN
6545-01-289-7095	6545-01-521-8528
6605-01-326-1654	6545-01-521-8528
6695-00-118-2918	6545-01-521-8522
	6545-01-521-8528
6840-00-142-8965	6545-01-521-8528
6840-01-284-3982	6545-01-521-8511
6850-00-985-7166	6545-01-521-8502
	6545-01-521-8511
	6545-01-521-8522
	6545-01-521-8528
	6545-01-521-8530
6910-01-515-4278	6545-01-521-8519
7210-00-715-7985	6545-01-521-8514
7610-01-301-5255	6545-01-521-8522
7610-01-301-5257	6545-01-521-8528
7610-01-301-5259	6545-01-521-8530
8105-01-521-0403	6545-01-521-8522
8315-00-787-8000	6545-01-521-8511
	6545-01-521-8514
8415-01-192-2357	6545-01-521-8522
	6545-01-521-8528
8520-00-551-0375	6545-01-521-8522
8520-00-782-3554	6545-01-521-8530
8925-01-008-0960	6545-01-521-8530
8935-00-082-5667	6545-01-521-8530
9535-01-201-7014	6545-01-521-8530
9920-01-191-3434	6545-01-521-8522
	6545-01-521-8528
	6545-01-521-8530

